




It's 99.9999% deadly.  
Just not to you.



in Canada

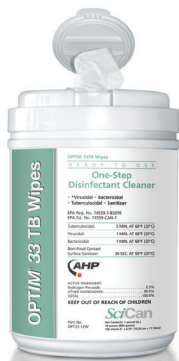

Virucidal. Bactericidal. Tuberculocidal. Just not harmful to you or your patients. OPTIM<sup>®</sup> disinfecting wipes kill germs on surfaces fast – up to 10 times faster than other leading cleaners. OPTIM cleans supreme using a patented formulation based on Hydrogen Peroxide that has virtually no odor. Also, the solution readily biodegrades into water and oxygen after disinfection. So OPTIM is eco-friendly and people friendly. In fact, it's really only germs that aren't too fond of it. Take control, because the stakes are too high.™

For more information, please visit [HealthFirst.com](http://HealthFirst.com) or call 800 331 1984

SciCan Dental  
Your Infection Control Specialist™



HealthFirst  
for your practice, your patients, your planet.



## OPTIM 33TB WIPES – EXCEPTIONAL CLEANING & DISINFECTION CAPABILITIES

- One step disinfectant cleaner – fast & convenient
- AHP – patented technology designed to deliver better cleaning and faster disinfection results (active ingredient 0.5% hydrogen peroxide)
- Hydrogen peroxide does not leave behind active residue which may lead to the development of resistant strains of microorganisms



### FAST CONTACT TIME:

- 1 minute bactericidal & virucidal claim
- 30 second sanitizer
- 5 minute TB claim

## SUPERIOR ENVIRONMENTAL AND SAFETY PROFILE

- No strong chemical odor
- Active ingredient breaks down into oxygen & water within 1 minute of disposal into sewage system
- Non-flammable
- Non-irritating to skin & eyes
- 0 HMIS hazard rating\*, non-toxic, EPA Cat. 4, the lowest toxicity level on EPA toxicity category

Active Ingredient	EPA Toxicity Category	Signal Word Required
AHP	Cat.4	NONE
Quats+Low % Alcohol	Cat.3	CAUTION
Quats+High % Alcohol	Cat.2	WARNING
Phenolics	Cat.1	DANGER

\* Refer to HMIS rating on MSDS & compare with our competitors' wipes

## SURFACE DISINFECTANT CONTACT TIME\*\*

Product Name	Active Ingredient	Mycobacteria												
		TB	Non-enveloped virus			Enveloped virus			Bacteria					
			Poliovirus	Norovirus	Rotavirus	HIV	HBV (Hep.B)	HCV (Hep.C)	MRSA	VRE	Influenza A2	E.coli	Pseudomonas	
OPTIM 33 TB	AHP	5	1	1	1	1	1	1	1	1	1	1	1	
Birex SE	Phenolics	10	10			10	10		10		10	10	10	
Prospray	Phenolics	10	10			1	10	10	10		10	10	10	
Caviwipes	Quats-alcohol	3				2	2	2	2	2			3	
Opticide 3	Quats-alcohol	3	3			3	3	3	3	3	3	3	3	
Sanicloth Plus	Quats-alcohol	3				2	2	2	2	2		2	3	
Lysol IC	Quats-alcohol	10	10			10							10	

\*\* Contact time in minutes. Contact time was obtained from publicly available material & is valid when table was constructed. Information will be updated if there are changes in claims. If blank, no published information was available.

### ORDERING INFORMATION FOR OPTIM 33TB

Ready-To-Use Pre-saturated Wipes:

**3600** RTU Wipes 6" x 7" (12 x 160 sheets/case)

Dry Wipes:

**3603** STAT-Wipes Canister & Roll of Dry Wipes (12 x 160 sheets /case)

Liquid:

**3602** 1 case of 12 x 32 oz.

**3601** 1 case of 4 x 1 gallon

Distributed by HealthFirst • 22316 70th Ave W Ste A Mountlake Terrace, WA 98043 • Toll Free 800 331 1984 • FAX 425 775 2374 • HealthFirst.com