Sharps-A-Gator™

Sharps Container

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en

To Lock the Slide: Close slide fully to side until it snaps locked.

Note: Slide may be temporarily closed when not in use to prevent accidental contact with contents by not sliding completely to closed, locked position.

Precautions/warnings

- Always keep used container in upright position.
- Never put hands into container opening.
- Use the proper size container for the items and volume being disposed.
- Do not place anything on top of container.
- Do not strike, tap, shake or manipulate container for any reason, i.e. to settle contents.
- Never hold or carry used container below recommended fill line.
- Use both handles on container when moving or lifting for disposal.
- Never hold or carry container by the lid.
- Do not hold or carry container close to the body.
- Never attempt to disassemble container. Exposure may occur to biohazardous materials and cause percutaneous or other injuries.
- Use container with appropriate Covidien hardware that fits with the Covidien container of choice. See product catalog for cross-reference.
- For single use only. Dispose of used container according to facility protocol.

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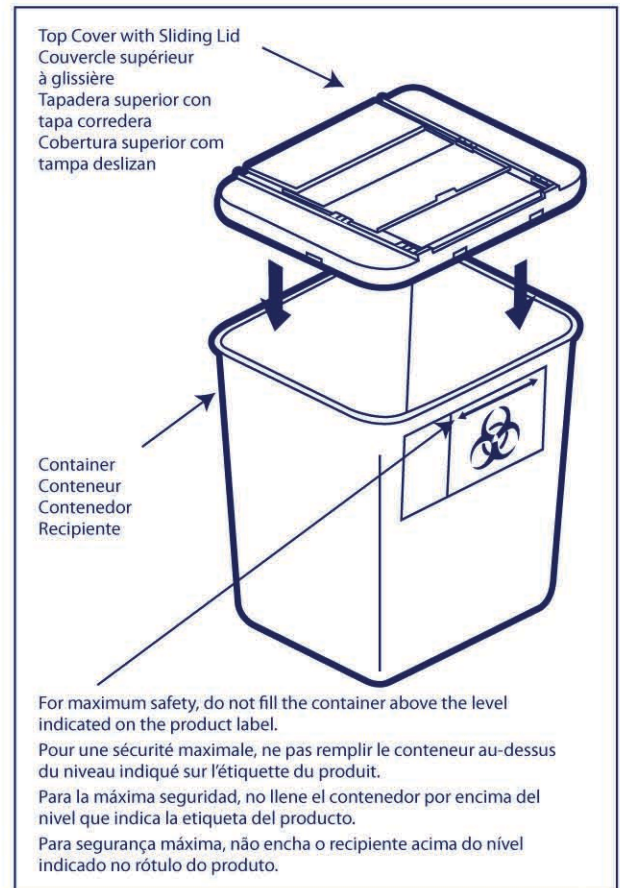
This product is safe for autoclaving or incineration.

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Single use

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

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IFU No. PT00033501

COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG.

Other brands are trademarks of a Covidien company.

© 2011 Covidien.

Covidien Ilc, 15 Hampshire Street, Mansfield, MA 02048 USA.

www.covidien.com

REV 3/2016