SharpSafety™

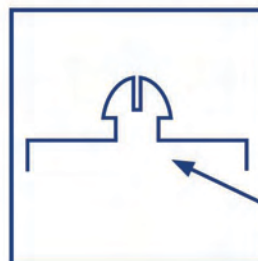
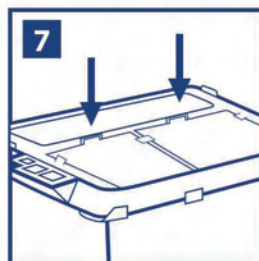
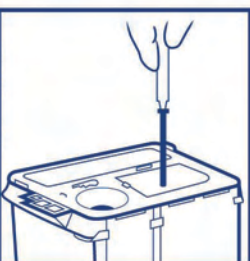
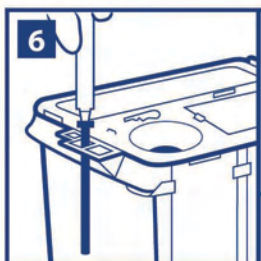
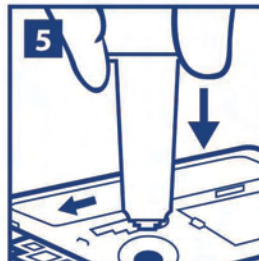
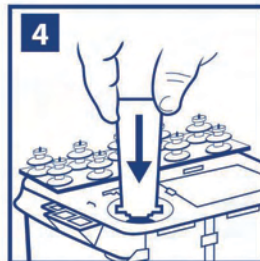
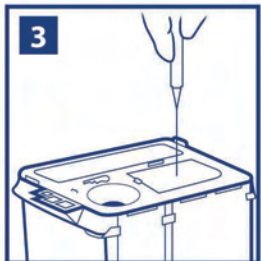
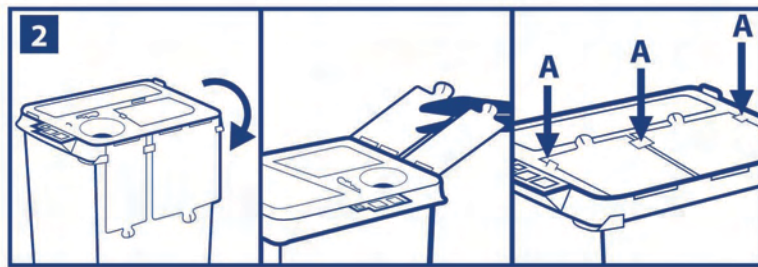
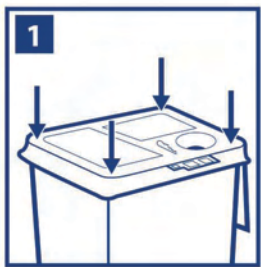
Phlebotomy Sharps Container

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en

Assembly and Usage Instructions

1. Snap lid on container by pressing down on all four corners.
Important – Make sure top is completely pushed down onto container.
Check that the container has suffered no shipping damage before placing into use.
2. When container is in use, covers should be secured by pressing onto side locking tabs located on the rim of the lid. If desired, separate covers with scissors to allow separate closure. For Temporary Closure – snap covers down to engage temporary locking tabs. Make sure both sides of covers are engaged. Listen for clicks.
(A = Locking tabs)
3. Use one hand to drop waste in container. Do not dispose of more than one sharp at a time.
4. To Use Autodrop Funnel – Use one hand to disengage needle. Insert needle holder vertically into funnel. Push down until needle is released.
5. To Use Unwinder – Use one hand on holder to insert needle into circular opening and slide to proper slot. Unscrew needle and slide holder back to the circular opening allowing needle to fall into container.
6. To Use Recapper – Remove cap and place in recapper. After use, use one hand to put needle in cap and dispose of in the appropriate container.
7. For Final Disposal – Lock covers by placing locking tabs in slots on the lid.
 - The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three-quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

Carrying/lifting instructions

Never hold or carry used container below recommended fill line. If handles are present, use both handles on container when moving or lifting for disposal. Use additional caution with containers that do not have handles.

Precautions/warnings

- Always keep used container in upright position.
- Never put hands into container opening.
- Use the proper size container for the items and volume being disposed.
- Do not place anything on top of container.
- Do not strike, tap, shake or manipulate container for any reason, i.e. to settle contents.
- Never hold or carry used container below recommended fill line.
- Use both handles on container when moving or lifting for disposal.
- Never hold or carry container by the lid.
- Do not hold or carry container close to the body.
- Never attempt to disassemble container. Exposure may occur to biohazardous materials and cause percutaneous or other injuries.
- Use container with appropriate Covidien hardware that fits with the Covidien container of choice. See product catalog for cross-reference.
- For single use only. Dispose of used container according to facility protocol.

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© 2011 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG.
Other brands are a trademark of a Covidien company.