# SharpSafety™

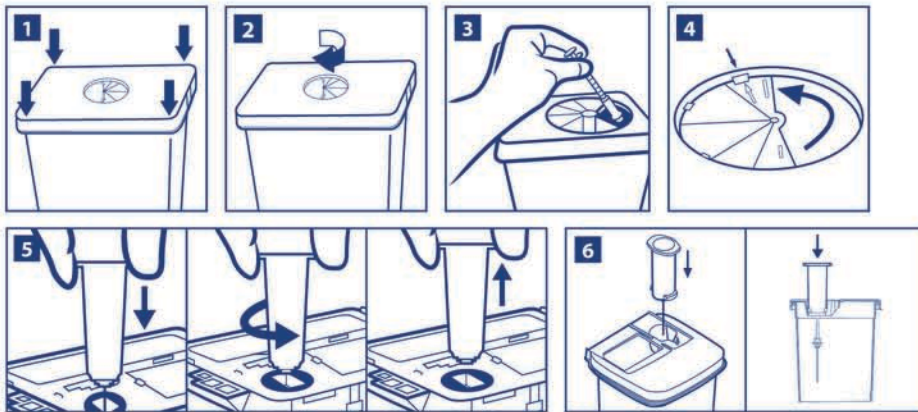
## Rotor Lid Sharps Container

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### en

#### Assembly

1. Snap lid on container by pressing down on all four sides. Listen for snaps. Please check that the container has suffered no shipping damage before placing into use.
2. Open rotor by turning clockwise until the opening is the desired size.

#### Stabilization

To maintain container upright and stable, install container in appropriate Covidien hardware. Check with your Covidien account representative for product catalog and cross-references.

#### Usage Instructions

3. To dispose of waste - Use one hand to drop item in the largest opening of the container. Ensure opening path is clear. Do not dispose of more than one item at a time.

#### Filling/no overfilling

Do not fill container above recommended fill line or when three quarters full if fill line is not present.

The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

#### To temporarily close door

Rotor may be closed temporarily when not in use to prevent accidental contact with contents. Ensure opening path is clear and turn rotor counterclockwise.

#### For final closure and locking

4. To lock container for final closure, turn rotor counterclockwise to the locking position by lining up arrows on rotor and container lid. Turn until audible click is heard. This is the final lock position for final disposal.

#### Unwinder Instructions:

5. To Use Unwinder: Needles should not be removed unless required by a specific medical procedure. Use one hand to insert needle into circular opening and slide to proper slot. Unscrew needle and slide holder back to the circular opening allowing needle to fall into container.

Please note: Not recommended for use.

The Federal Needlestick and Prevention Act does not promote the use of needle unwinders.

CSA standard 4.2.9 states that containers shall not have notches or similar mechanisms for unwinding or removing needles from syringes.

OSHA Standard CPL 02-02-069 ...paragraph (d)(2)(vii)(A), which prohibits needle removal unless no alternative is feasible or it is required by a specific medical procedure, should be cited. If needle removal must be accomplished, the employee must be trained in the correct procedure as required by paragraph (g)(2)(vii)(F).

#### Funnel Instructions for Auto Drop System:

6. To Dispose Of Needle -Using one hand, push holder straight down into the funnel of container to release needle and disposal thread.

### Carrying/lifting instructions

Never hold or carry used container below recommended fill line. If handles are present, use both handles on container when moving or lifting for disposal. Use additional caution with containers that do not have handles.

### Precautions/Warnings

- Always keep used container in upright position.
- Never put hands into container opening.
- Use the proper size container for the items and volume being disposed.
- Do not place anything on top of container.
- Do not strike, tap, shake or manipulate container for any reason. I.e. to settle contents.
- Never hold or carry used container below recommended fill line.
- Where handles are present, use both handles on container when moving or lifting for disposal.
- Do not hold or carry container by the lid.
- Do not hold or carry container close to the body.
- Never attempt to disassemble container. Exposure may occur to biohazardous materials and cause percutaneous or other injuries.
- Use container with appropriate Covidien hardware that fits with the Covidien container of choice. See product catalog for cross-reference.
- For single use only. Dispose of used container according to facility protocol.

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© 2012 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG.
Other brands are trademarks of a Covidien company.