SharpSafety™

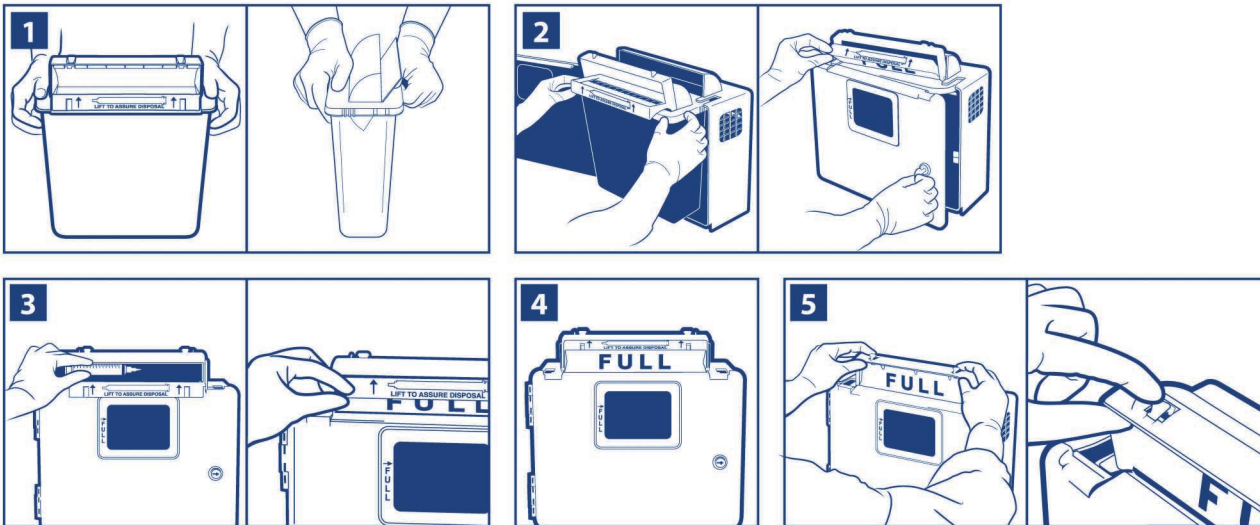
Safety In Room Sharps Containers

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en

Assembly

1. Snap lid on container by pressing down on all four sides. Listen for clicks. Important – Make sure top is completely pushed down onto container. Check that the container has suffered no shipping damage before placing into use.

Stabilization

2. To maintain container upright and stable, place container into appropriate Covidien wall enclosure or bracket, lift flap to close door and lock with key if it is a locking unit. Check with your Covidien account representative for product catalog and cross-references.

Usage Instructions

3. To dispose of waste - use one hand to place sharp in slot horizontally, making sure sharp falls into container. Lift to assure sharp drops. Do not dispose of more than one item at a time.

Filling/no overfilling

4. Container is full if door does not return to the open position or the "Full" indicator is visible. The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

For final closure and locking

5. To lock container for final closure and disposal – Lock container before removing from enclosure or bracket. To lock, push flap all the way closed and snap both locking tabs down. Listen for clicks.

Open enclosure or bracket and replace container. Replace container immediately. Use both handles on container when moving or lifting for disposal.

Carrying/lifting instructions

Never hold or carry used container below recommended fill line. If handles are present, use both handles on container when moving or lifting for disposal. Use additional caution with containers that do not have handles.

Precautions/warnings

- Always keep used container in upright position.
- Never put hands into container opening.
- Use the proper size container for the items and volume being disposed.
- Do not place anything on top of container.
- Do not strike, tap, shake or manipulate container for any reason, i.e. to settle contents.
- Never hold or carry used container below recommended fill line.
- Never hold or carry container by the lid.
- Where handles are present, use both handles on container when moving or lifting for disposal.
- Do not hold or carry container close to the body.
- Never attempt to disassemble container. Exposure may occur to biohazardous materials and cause percutaneous or other injuries.
- Use container with appropriate Covidien hardware that fits with the Covidien container of choice. See product catalog for cross-reference.
- For single use only. Dispose of used container according to facility protocol.

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© 2011 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG.
Other brands are trademarks of a Covidien company.