



A/ Instructions for closure of Mono and Duo Lids

1° step:

push on two opposite corners to firmly hook the lid on the body




2° step:

push on the other two opposite corners




3° step:

push on the edge of the short sides and insert the clips

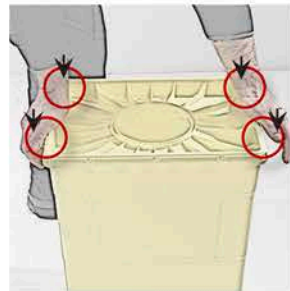


4° step:


push on the edge of the long sides and insert the clips

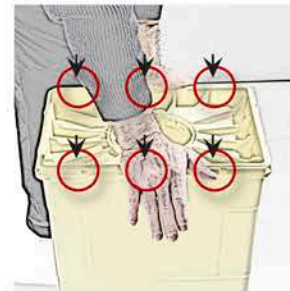

A sound ensures that the clip has been correctly inserted and the clip can be seen in the edge of the lid

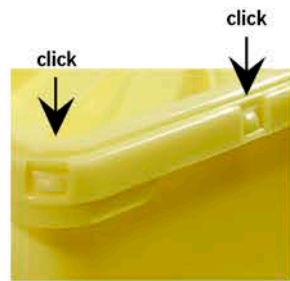


The lid is sealed on the perimeter of the body and the container is watertight thanks to the glue on the lid

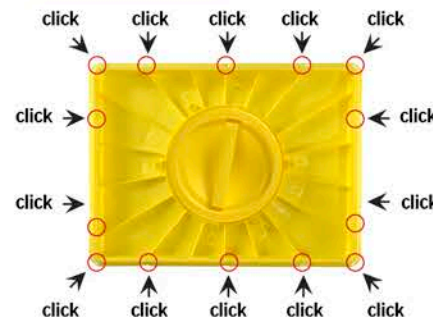

C/ Placement of the lid

Mono lid can be hooked to the body:

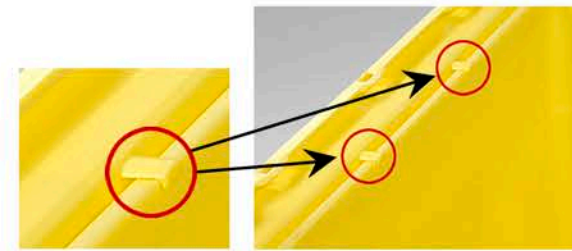

on the long side




on the short side



D/ Transport of the container

1/ With the central handle

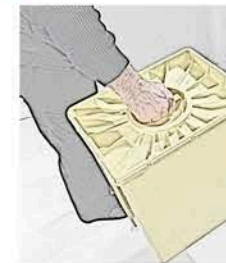

WARNING

With Lid Duo New:
Use the central handle only with screw lid in permanent closure DO NOT use the central handle of the screw lid in temporary closure


2/ With the side handles



B/ Instructions for closure screw lids Duo-N


1/ Insertion:

a) place the screw lid on position as indicated on the lid

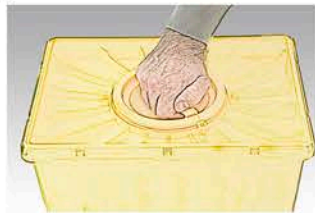

2/ Temporary closure:

Turn the screw lid to the left or to the right as indicated on the lid

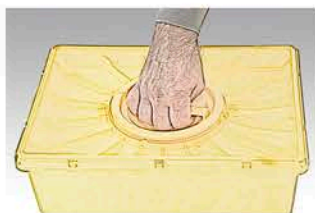

3/ Permanent closure


a) take off the screw lid



b) place the screw lid on the indicated position



c) Press the screw lid firmly until the clic of the definitive closure

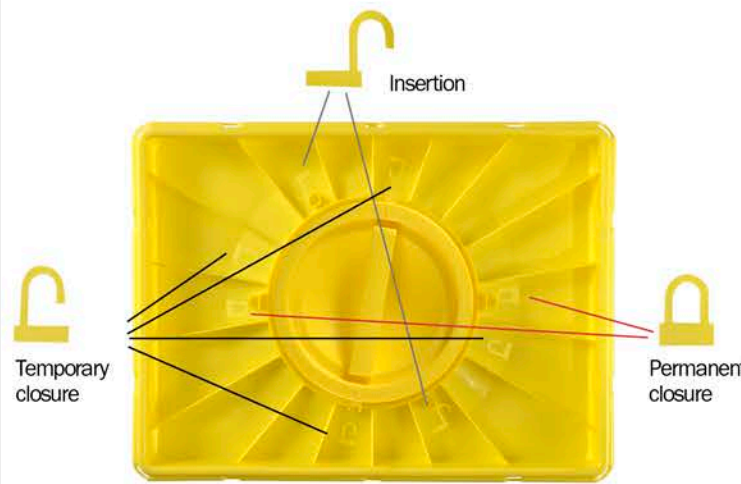

The container is watertight thanks to the glue on the screw lid



E/ Use of the trolley

1/

Insert the container placing the long sides on the rails of the trolley



2/

Insert the lid in the proper location under the handle



3/

Open and close the lid with the pedal



F/ Certificates:

Product compliant with NFX 30-505 norms (EVO - SC)

Product compliant to ADR (UN Certificate) (EVO - SC - SB)

Resistant to penetration in compliance with BS 7320 (British Standard) (EVO - SC - SB)